



# TZ MEDICAL

## NEPTUNE DISC DIRECTIONS FOR USE

1.



### Collecting Supplies

Have Neptune Disc, EZ Hold, and any other needed supplies near patient within reach of the sterile field. Using sterile technique, open the Neptune Disc packaging and transfer the disc onto the sterile field.

2.



### Preparation

Assess vital signs, access site, and digital pulses.

3.



### Application

Attach disc to EZ Hold, and position the disc directly above the femoral artery. Remove femoral sheath. Activate Neptune pad with a few drops of blood. DO NOT SATURATE PAD.

4.



### Pressure

Using the EZ Hold, apply proximal occlusive pressure for 3-4 minutes. Slowly release pressure to maintain semi-occlusive compression until hemostasis is achieved.

5.



### Neptune Removal

Gently roll back EZ Hold and disc. With free hand, press down lightly on the Neptune pad so that it detaches from the disc and remains on the access site.

6.



### Post-Hemostasis

Cover Neptune pad with dressing of choice. The Neptune pad can stay on for up to 24 hours. To remove, saturate pad with water and peel off gently.

