

COMFORT SHIELD CLEANING INSTRUCTIONS

The TZ Medical [Comfort Shield](#) and [URPro Drape](#) is made of strong, durable vinyl material that will withstand many uses; however, care must be taken when cleaning the product to keep it in good condition.

Washing:

1. **Prewash** to remove any caked-on material by rinsing with water and wiping with a nonabrasive cloth or sponge.
2. **Wash** the Comfort Shield or URPro Drape with a mild detergent (such as ENZOL® Fast-acting Enzymatic Cleaner) by briefly spraying and wiping down. Isopropyl alcohol (rubbing alcohol) may be used to spot clean only but **NOT** for soaking. Avoid abrasives such as brushes or scouring pads as they may scratch the surface.
3. **Rinse** off all cleaning agents. *Only clean the Comfort Shield or URPro Drape with chemicals which are indicated for cleaning vinyl surfaces.*
4. **Dry** using a cloth, air, or heat up (be sure the Comfort Shield or URPro Drape is not exposed to extreme heat)

Note: Store the Comfort Shield or URPro Drape in a dry, cool and well-ventilated place and store away from extremely high or low temperatures.

CAUTION:

- **DO NOT allow the Comfort Shield or URPro Drape to be exposed to extremely high or low temperatures.** If a Comfort Shield or URPro Drape is suspected to have been exposed to extremely high or low temperatures, inspect for visible damage (melted, deformed, or discolored areas).
- **DO NOT steam clean** or allow the Comfort Shield or URPro Drape to come into contact with steam as it can degrade the materials of the product.
- **DO NOT clean the Comfort Shield or URPro Drape with any chemical not specifically indicated for the cleaning of vinyl surfaces.**
- **DO NOT use a Comfort Shield or URPro Drape that has evidence of damage to any part of the product.**